
Glucose Meter Reference Design

*Author: Namrata Dalvi
Microchip Technology Inc.*

INTRODUCTION

This application note describes the basic glucose meter design using Microchip's PIC[®] 8-bit PIC16LF178X XLP device.

A glucose meter is a medical device used to determine the concentration of glucose in the solution. The glucose concentration is measured in units of milligram per decilitre (mg/dl) or millimole per litre (mmol/L), depending on the different regions.

The glucose meter is a key element of the Home Blood Glucose Monitoring (HBGM) device used by people with diabetes mellitus. The measurements can be taken multiple times in a single day.

PRINCIPLE OF GLUCOSE MEASUREMENT

The glucose meter determines the concentration of glucose in the solution. This application note focuses on the design of a glucose meter, including the necessary hardware design and software development.

Most glucose meters are based on electrochemical technology. They use electrochemical test strips to perform the measurement. A small drop of the solution to be tested is placed on a disposable test strip that the glucose meter uses for the glucose measurement. The two most common methods used in electrochemical measurement of glucose are the Colorimetric method and the Amperometric method.

Colorimetric Method

In this method, the typical sensors such as LEDs or photo sensors form the analog interface. These sensors are followed by a Transimpedance Amplifier (TIA) for the glucose concentration measurement in the solution. The Color Reflectance principle is used in this method to sense the color intensity in the reaction layer of the test strip by the photometry. The glucose meter generates a numerical value, that is a measurement of the glucose concentration present in the solution.


Amperometric Method

In this method, the electrochemical test strip contains a capillary that is used to draw in the solution placed at one end of the test strip. The test strip also contains an enzyme electrode containing a reagent such as Glucose Oxidase. Glucose undergoes a chemical reaction in the presence of enzymes and electrons are produced during the chemical reaction. These electrons (i.e., the charge passing through the electrode) are measured and this is proportional to the concentration of glucose in the solution. An ambient temperature measurement is also made in order to compensate for the effect of temperature on the rate of the reaction.

TEST STRIP WORKING PRINCIPLE IN AMPEROMETRIC METHOD

The majority of glucose meters are electrochemical and use the Amperometric method. Figure 1 illustrates the glucose meter test strip working principle.

FIGURE 1: GLUCOSE METER TEST WORKING STRIP PRINCIPLE

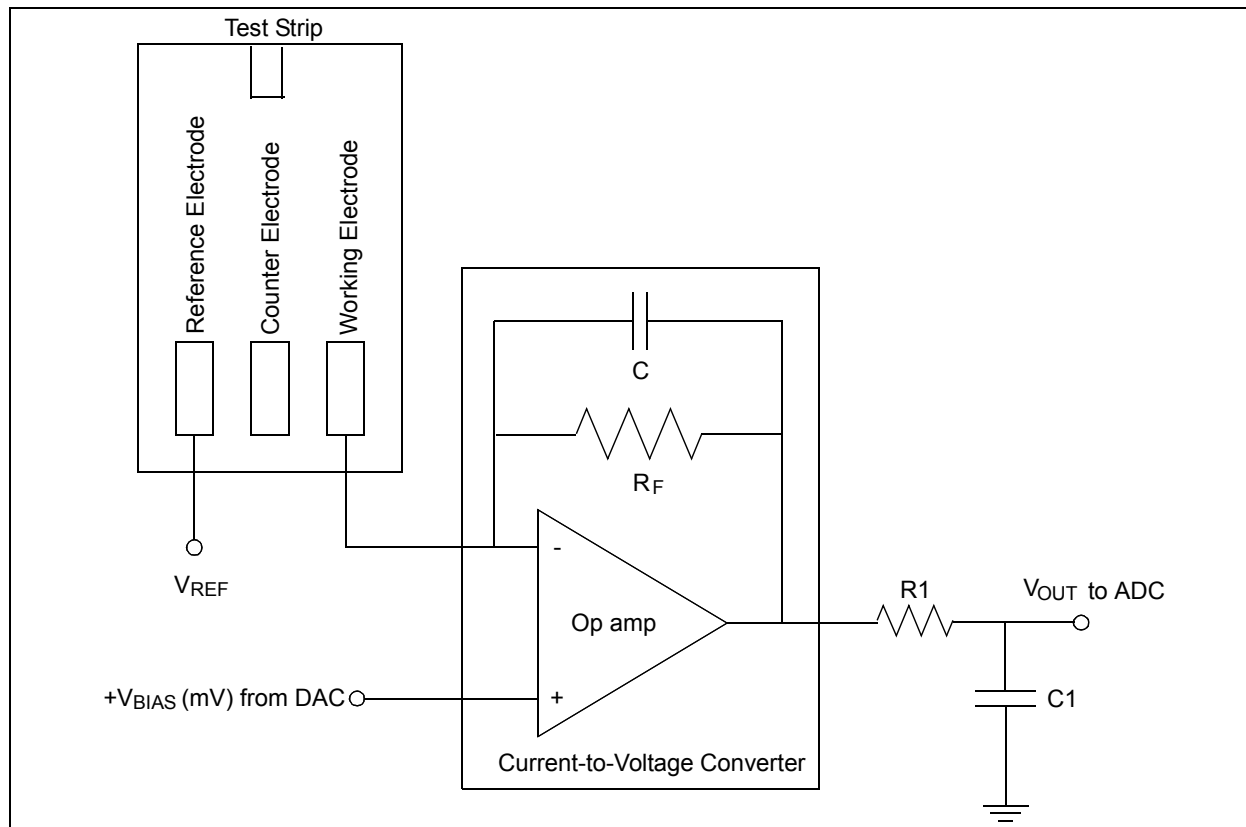

The test strip forms the main biochemical sensor where the sample of solution is placed. The test strip has the following electrodes:

- **Working electrode:** Electrons are produced here during the chemical reaction. This electrode is connected to the current-to-voltage amplifier.
- **Reference electrode:** Held at a constant voltage with respect to the working electrode to push the desired chemical reactions.
- **Counter electrode:** Supplies current to the working electrode.

Most of the glucose meter designs use only two electrodes, reference electrode and working electrode.

A precise reference voltage (V_{REF}) is applied to the reference electrode and a precise bias voltage (V_{BIAS}) is applied to the op amp. This way the precise potential difference is maintained across the working electrode and the reference electrode. This voltage is the stimulus which drives the test strip's output current. The magnitude of the output current is then used to calculate the number of electrons produced.

The solution sample is placed on the test strip and the reaction of the glucose with the enzyme takes place. Electrons are generated during the chemical reaction. Flow of electrons will correspond to the flow of current through the working and the reference electrode. This current will change according to the glucose concentration. The current is measured using a transimpedance amplifier (current-to-voltage converter) for the measurement with an Analog-to-Digital Converter (ADC).


The output of the transimpedance amplifier will be seen as a variation in the voltage with varying glucose concentrations in the solution.

DIGITAL IMPLEMENTATION OF GLUCOSE METER USING THE PIC16LF178X DEVICE

This section details the hardware design and the software development of the glucose meter using the Amperometric method.

Figure 2 illustrates the block diagram of a typical glucose meter. The glucose meter can be implemented using the PIC16LF178X device.

FIGURE 2: GLUCOSE METER BLOCK DIAGRAM

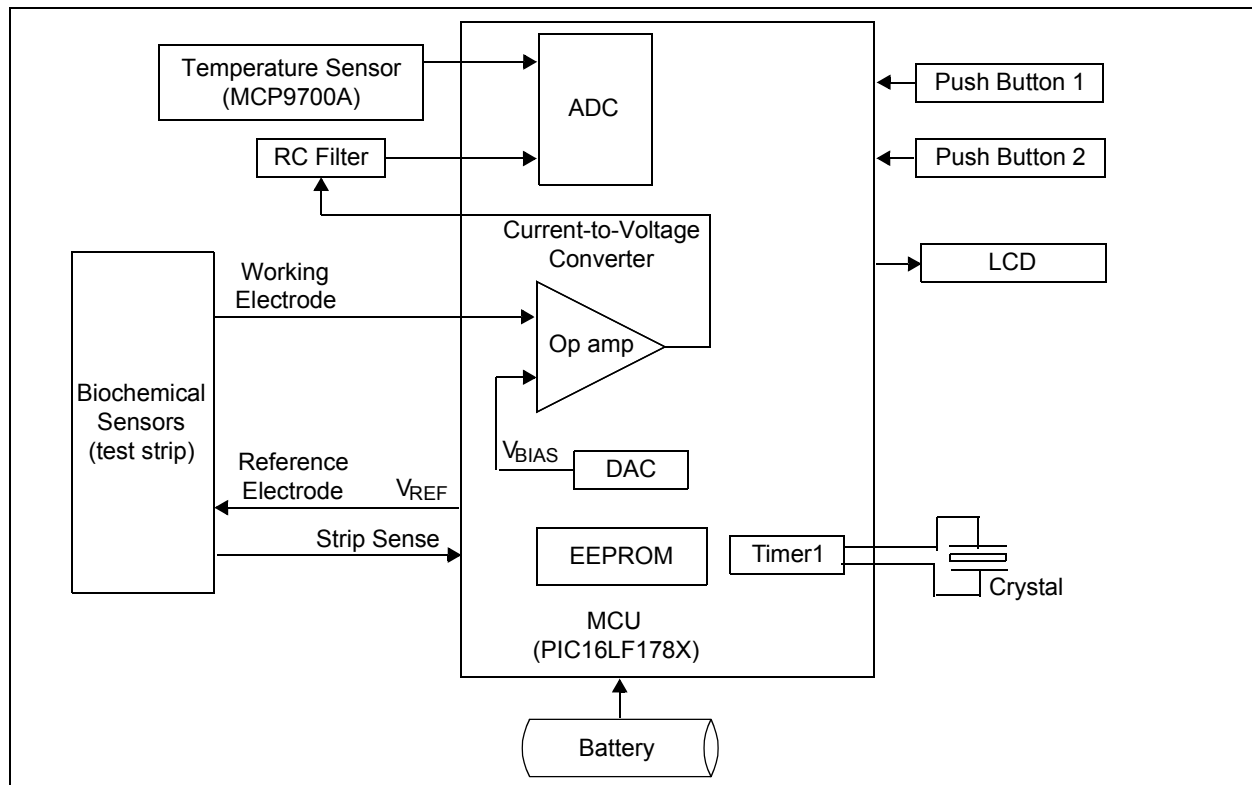

The following are the features of the PIC16LF178X device and some of the peripherals in its integrated measurement engine:

- Extreme low-power (XLP) operation
- Two op amps
- 2x8-bit DAC
- 12-bit Successive Approximation (SAR) ADC, up to 11 channels
- Internal EEPROM
- Inter-Integrated Circuit (I²C™)
- 16-bit Timer1

When the solution sample is placed on the test strip, glucose undergoes a chemical reaction and electrons are produced. The flow of electrons (i.e., the current flowing through the working electrode) can be measured. This current will change according to the glucose concentration. The current is measured with the help of the current-to-voltage conversion using the internal op amp of the PIC16LF178X device and the filtering of high-frequency signals. The filtered signal is fed to the 12-bit ADC module of the device.

The PIC16LF178X device starts capturing the voltage at the ADC channel after about 1.5 seconds of placing the solution sample. An average of about 2048 ADC readings is taken. This average value is substituted into the regression equation.

The glucose concentration is determined using this regression equation and the value is displayed on the LCD in units of mg/dl or mmol/l. Up to 32 glucose readings can be stored in the internal EEPROM and can be viewed later on the LCD. The power to the Glucose Meter Demo Board can be supplied from the on-board lithium battery (3V 225 mA CR2032).

Note: The time to start capturing the ADC values (i.e., one second to 1.5 seconds) and the number of ADC readings taken should be modified to match the type and characteristics of the test strip to be used.

GLUCOSE METER REFERENCE DESIGN HARDWARE

This section provides the hardware design details of the glucose meter.

Design Specifications

- Glucose measurement range: 20 mg/dl to 600 mg/dl (1 mmol/l to 33 mmol/l)
- Test result is displayed within five seconds
- Automatic storage of last 32 glucose readings with date and time stamp
- No test strip coding: Generic regression equation will be implemented and can be modified based on the test strip characteristics

Hardware Design Features

- Number of boards: Single board
- PIC16LF178X Device: 28-pin device
- Debug/program: In-Circuit Serial Programming™ (ICSP™) connection provided
- Monitoring/indication: Test points
- Test strip connection: Terminals/connector provided
- RTCC (using internal Timer1): Date and time stamp for the glucose meter
- Internal EEPROM:
 - Record last 32 readings of the glucose meter
 - Store any parameters or calibration data related to the test strip
- LCD:
 - Display the solution glucose content in units of mg/dl or mmol/l
 - Display the messages: "Insert Test Strip", "Strip Inserted Place The Sample", "Faulty Test Strip"
- Test strip sensing: To detect insertion of the test strip
- Temperature sensing: Provision given to consider temperature variation for the glucose calculations. If relation between the temperature and the glucose concentration is known, it can be incorporated in the regression equation to take care of any changes in the glucose concentration due to temperature variations.
- Battery voltage sensing: Battery health sensing and low battery indication
- Two push buttons: To read previously stored data on the LCD, and to set date and time
- Battery: CR2032: 3V, 225 mAH lithium battery

GLUCOSE METER FIRMWARE IMPLEMENTATION

Firmware Features

- Sensing of test strip current using the internal op amp, DAC, and ADC of PIC16LF178X
- Captures ADC readings after the test strip is inserted and checks for a rise above 450 mV
- Starts recording the ADC readings, 1.5 seconds after the test sample is placed on the test strip and calculates the average
- Calculates the glucose concentration using the regression equation and the average ADC reading


Firmware modules are available for the following features:

- LCD interface and display routines
- Configuration of op amp
- Configuration of DAC
- Store glucose readings into the internal EEPROM
- Reading the ADC channel
- Calculation of glucose concentration
- Implementation of the RTCC using Timer1 for time-stamping

Firmware Flowchart

Figure 3 illustrates the firmware flowchart.

FIGURE 3: FIRMWARE FLOWCHART

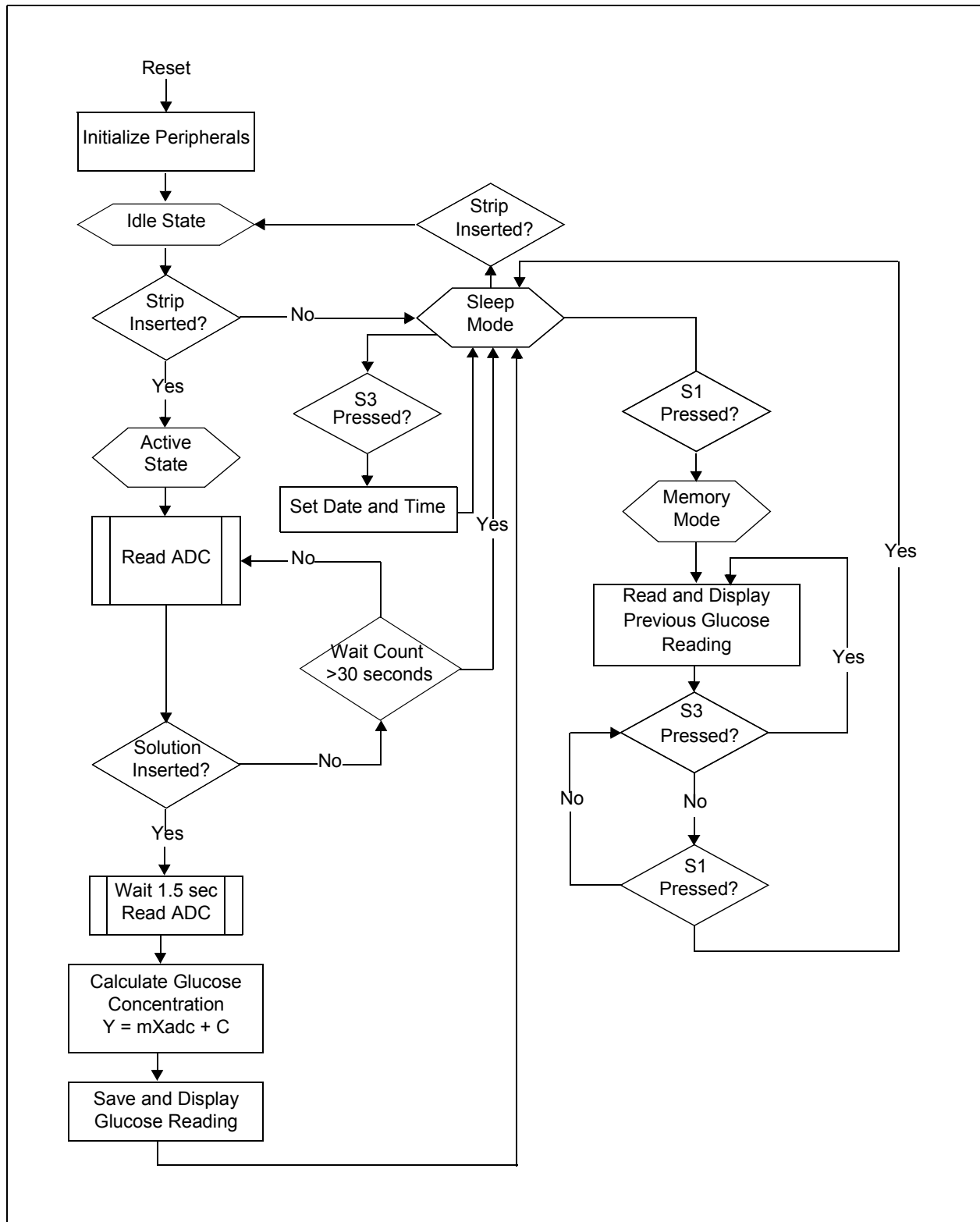

AN1560


DAC Configuration

The voltage reference of the DAC is connected to the internal Fixed Voltage Reference (FVR) buffer 2, configured for the 2.048V. The DAC output voltage is set to be 400 mV.

Op Amp Module Configuration

As shown in Figure 4, non-inverting input channel of the op amp is connected to the DAC output set at 400 mV. The inverting terminal of the op amp is connected to the working electrode. The current-to-voltage converter is formed with the help of the external resistor and the capacitor. The output of op amp is connected to the ADC channel of the PIC16LF178X device.

FIGURE 4: OP AMP CONFIGURATION

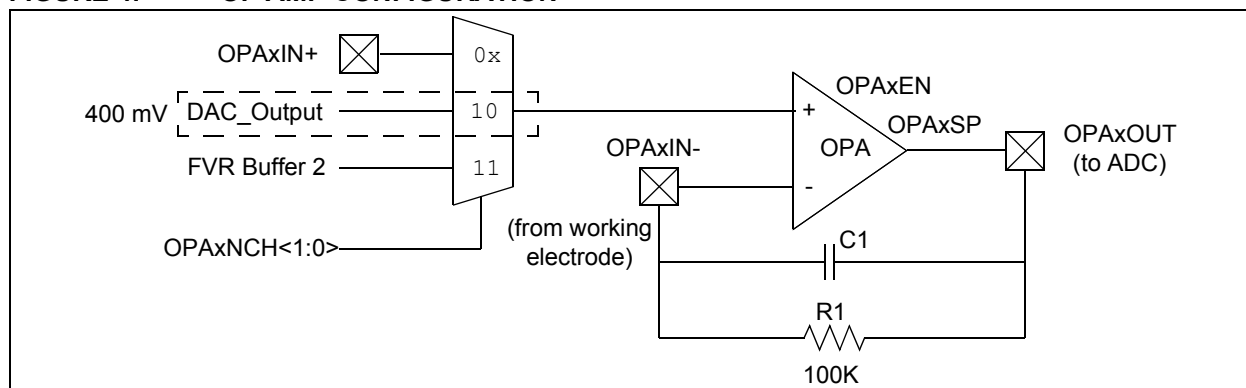

ADC Configuration

The op amp output (i.e., current-to-voltage converter output) is measured with ADC channel 0 (RA0/ AN0). ADC channel 3 (RA3/ AN3) is used to measure the battery voltage to indicate the low battery condition. Output of the temperature sensor (MCP9700A) is connected to the ADC channel 8 (RB2/ AN8) to read the temperature.

EEPROM

Glucose readings are stored in the internal EEPROM. During Sleep mode if the switch S1 is pressed, the PIC16LF178X device enters Memory mode and the stored glucose reading is displayed on the LCD. To view the previous glucose readings, the switch S3 has to be pressed. Press the switch S1 again to exit Memory mode.

LCD Interface

A 16 x 2 character LCD "NHD_C0216CZ-FSW-FBW-LCD" from NHD is used for displaying the glucose reading and text messages. Power to the LCD is cut off during Sleep mode by controlling the Vss of LCD through the port pin of the microcontroller.

RTCC

Timer1 along with the external 32.768 kHz watch crystal is used to implement the RTCC. The current date and time can be set for the RTCC using switches S1 and S3.

Determining Regression Equation

The following steps were used to determine the regression equation for the design of the Glucose Meter:

1. Using solutions with different glucose concentrations, the voltage profiles at the ADC output were captured.
2. Moving average of about 2048 ADC readings were taken, approximately 1.5 seconds after the sample was placed on the test strip. The average ADC reading was found to give a linear variation with respect to the glucose concentration of various test solutions (multiple ADC readings at short intervals can be taken and averaged to get more accurate results).
3. Determine the best fit equation using the poly-fit function in MATLAB®, regression feature in Microsoft® Excel or any other math tool. For example, a linear equation yielded good results, and was selected. See Equation 1.

EQUATION 1: GLUCOSE REGRESSION EQUATION

$$Y = mX + C$$

Where,

Y = Glucose concentration in mg/dl

m = Slope

X = Average ADC reading of op amp output voltage

C = Constant

GLUCOSE METER CURRENT CONSUMPTION

The current consumption of the glucose meter in Active mode is approximately 1.1 mA, and it consumes 3 μ A during Sleep mode. The glucose meter will be in Sleep mode most of the time. We can consider Sleep mode time to be approximately 99.5%. The average current consumption of the glucose meter can be calculated as shown in Equation 2.

EQUATION 2: AVERAGE CURRENT CONSUMPTION

$$0.995 \times 0.003 + 0.005 \times 1.1 = 0.008485 \text{ mA}$$

(i.e., 8.485 μ A)

Expected battery life for the 3V, 225 mAHr coin cell is 26517 hours (i.e., 3.03 years).

DEMONSTRATION OF GLUCOSE MEASUREMENT USING THE GLUCOSE METER BOARD

This section describes the process involved in the measurement of glucose using the Glucose Meter Demo Board. For the test purpose, a standard test solution with a known concentration of glucose can also be used.

Figure 5 illustrates the front view of the Glucose Meter Demo Board. Figure 6 illustrates the rear view of the Glucose Meter Demo Board.

The current date and time for the glucose meter can be set using switches S1 and S3.

FIGURE 5: GLUCOSE METER DEMO BOARD (FRONT VIEW)


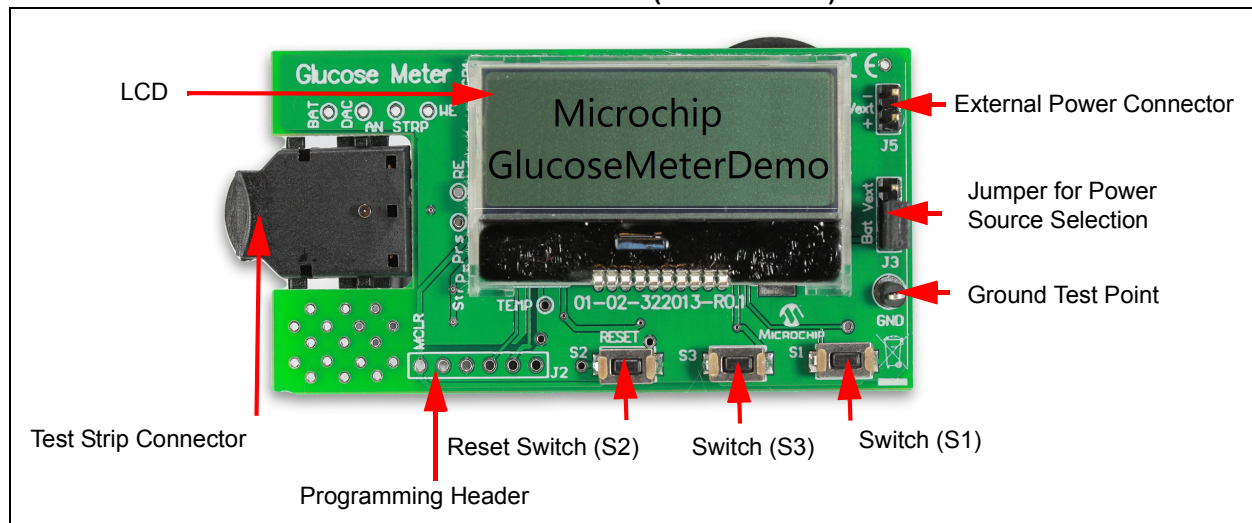


FIGURE 6: GLUCOSE METER DEMO BOARD (REAR VIEW)

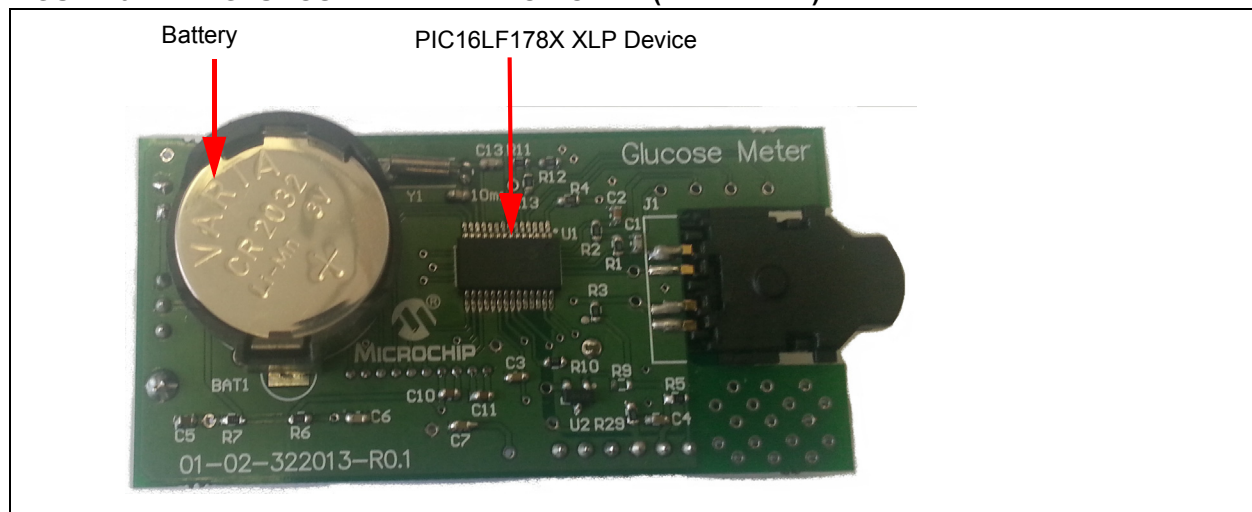


AN1560

Measuring Glucose using Glucose Meter

Perform the following steps for measurement of the solution glucose using the Glucose Meter Demo Board:


1. Power-up the Glucose Meter Demo Board, or press the switch S2 to reset the demo board. The start-up display will be as shown in [Figure 7](#).

FIGURE 7: GLUCOSE METER DEMO BOARD AFTER RESET

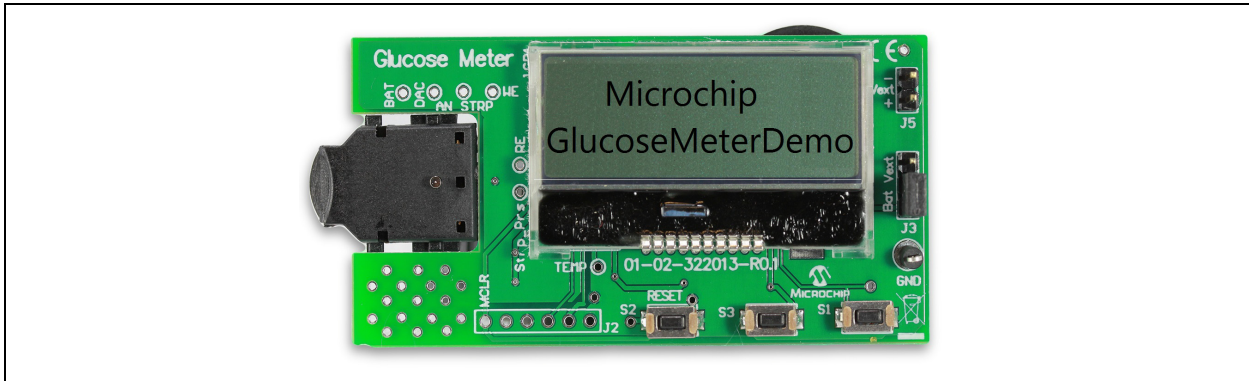


2. After about two seconds, the "Insert Test Strip" message will be displayed, see [Figure 8](#).

FIGURE 8: GLUCOSE METER DEMO BOARD WAITING FOR TEST STRIP INSERTION

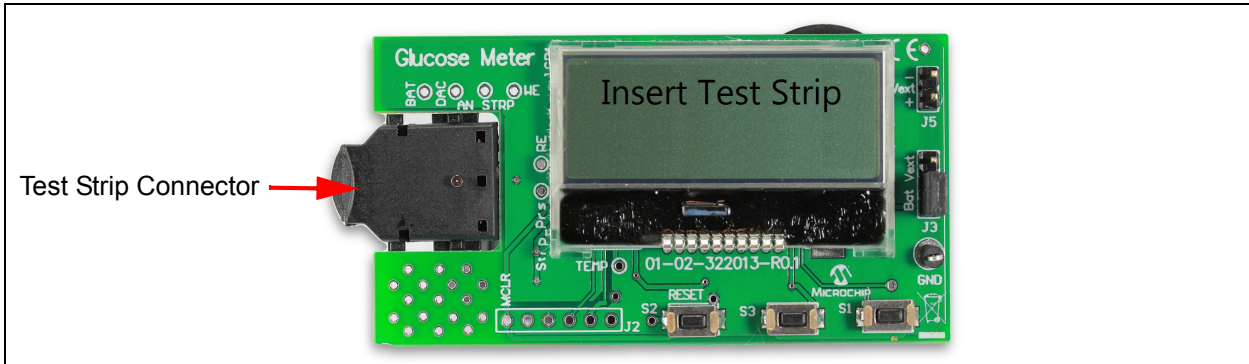


3. Insert the test strip into the test strip connector. When the test strip is inserted, the "Strip Inserted Place The Sample" message will be displayed, see [Figure 9](#).

FIGURE 9: GLUCOSE METER DEMO BOARD WAITING FOR SOLUTION SAMPLE

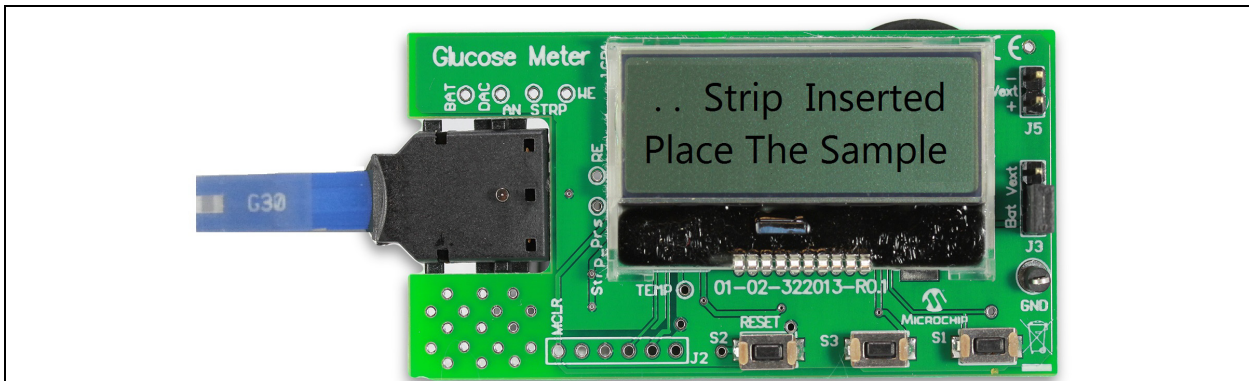


- Place the solution sample or standard test solution on the test strip, see [Figure 10](#).

FIGURE 10: GLUCOSE METER WITH STANDARD TEST SOLUTION PLACED ON TEST STRIP

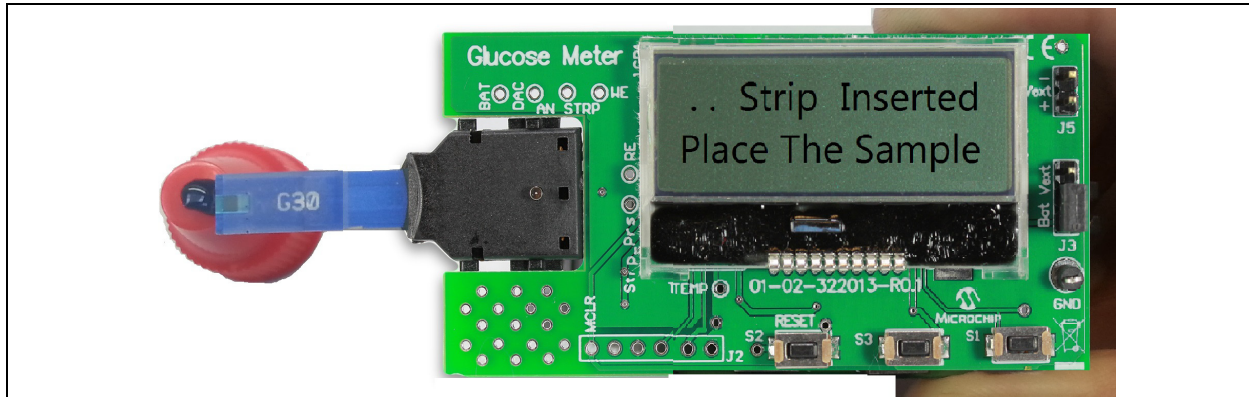

- The glucose meter starts capturing the output voltage profile of the transimpedance amplifier, and then calculates the glucose content in the solution.
- The concentration of glucose will be determined in 4 to 5 seconds, and displayed in units of measurement (mmol/l with time in hh:mm (24 hour) format and mg/dl with the date in mm/dd/yy format), see [Figure 11](#). The solution glucose reading will be saved in the internal EEPROM of the glucose meter.

FIGURE 11: GLUCOSE METER DEMO BOARD AFTER CALCULATING THE GLUCOSE CONCENTRATION IN THE SOLUTION

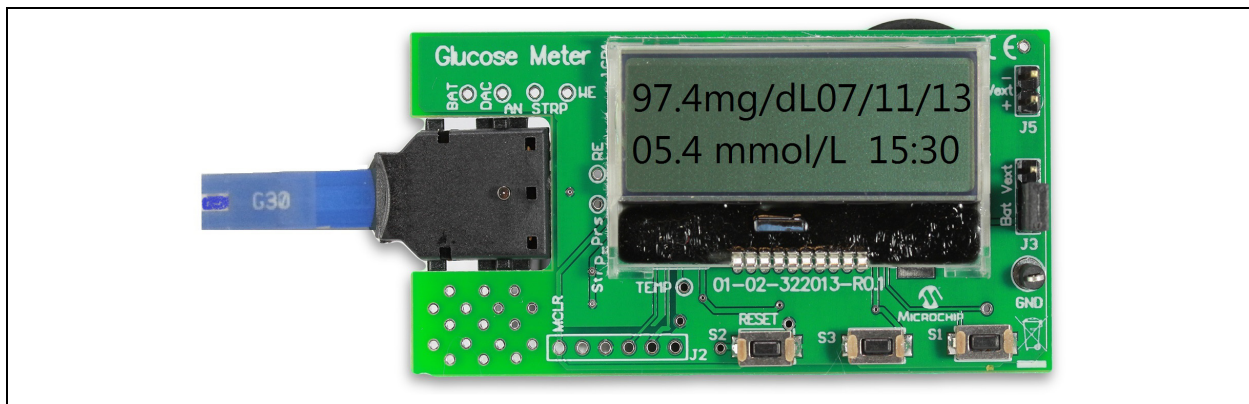

- Remove the test strip after the glucose measurement. The PIC16LF178X device will enter Sleep mode after removing the test strip.
- To measure another solution glucose reading, insert a new test strip in the test strip connector. The PIC16LF178X device will wake up from Sleep mode, then repeat Step 4 through Step 7 to measure the concentration of solution glucose.

AN1560


Setting Date and Time for the Glucose Meter

Perform the following steps to set the current date and time for RTCC of the glucose meter:

1. Press the switch S3 to set the current date and time for the RTCC. Enter date in mm/dd/yy and time in hh:mm (24 hour) format, see [Figure 12](#).


Note: To set the month as November (i.e., 11th month), press switch S1 once to set “1” as the first digit, and then press switch S3 to move to the next field entry. Press switch S1 once to set “1” as the second digit, and then press switch S3 to move to the next field entry (i.e., date). Repeat the procedure to set date, year, hour and minute for the RTCC. If the field to be set is zero, then press switch S3 directly to navigate to the next field entry.

FIGURE 12: GLUCOSE METER DEMO BOARD FOR SETTING DATE AND TIME

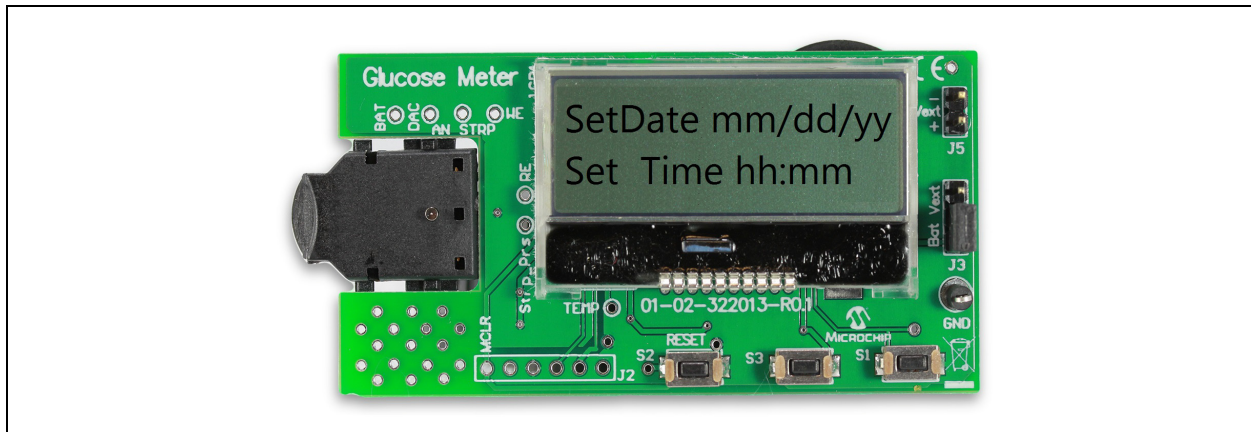

2. LCD will display the set date and time (for example, the date is set as 11th July 2013 and time is 3:20 p.m), see [Figure 13](#).

FIGURE 13: GLUCOSE METER DEMO BOARD AFTER DATE AND TIME SET

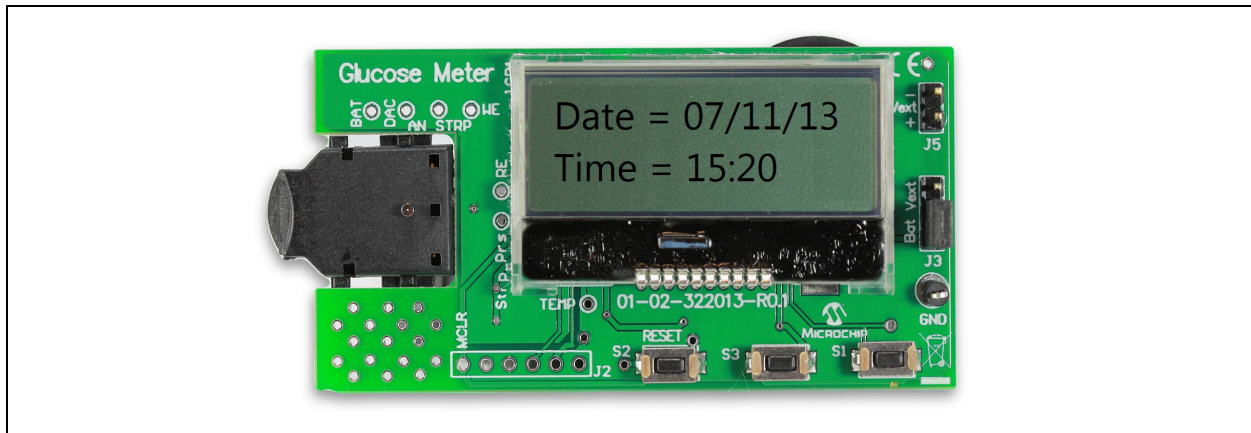

CONCLUSION

The glucose measurements are affected by the external factors such as temperature, humidity, altitude and so on, because the rate of the enzyme reaction depends on these and other factors. In addition, test strips with different chemistries will require variations in the regression equation determined using MATLAB or Microsoft[®] Excel. These factors must be considered when designing a glucose meter to use any particular test strip.

The PIC16LF178X MCU is equipped with op amp, 12-bit ADC, DAC and EEPROM, which makes a great combination for this type of battery operated application needing precision measurement and lower current consumption.

As this application note demonstrates, the Microchip PIC16LF178X MCU can be used to implement a flexible and low-cost glucose meter design.

REFERENCES

"PIC16(L)F1784/6/7 28/40/44-Pin 8-Bit Advanced Analog Flash Microcontrollers" (DS41637).

APPENDIX A: CURRENT-TO-VOLTAGE CONVERTER OUTPUT WITH VARIOUS GLUCOSE CONCENTRATIONS

This appendix provides the scope captures of the current-to-voltage converter output which is measured with ADC to determine the concentration of the glucose. Figure A-1 illustrates the current-to-voltage converter output with high control solution for the glucose reading of 17.5 mmol/l (i.e., 315 mg/dl).

FIGURE A-1: CURRENT-TO-VOLTAGE CONVERTER OUTPUT WITH CONTROL SOLUTION HIGH


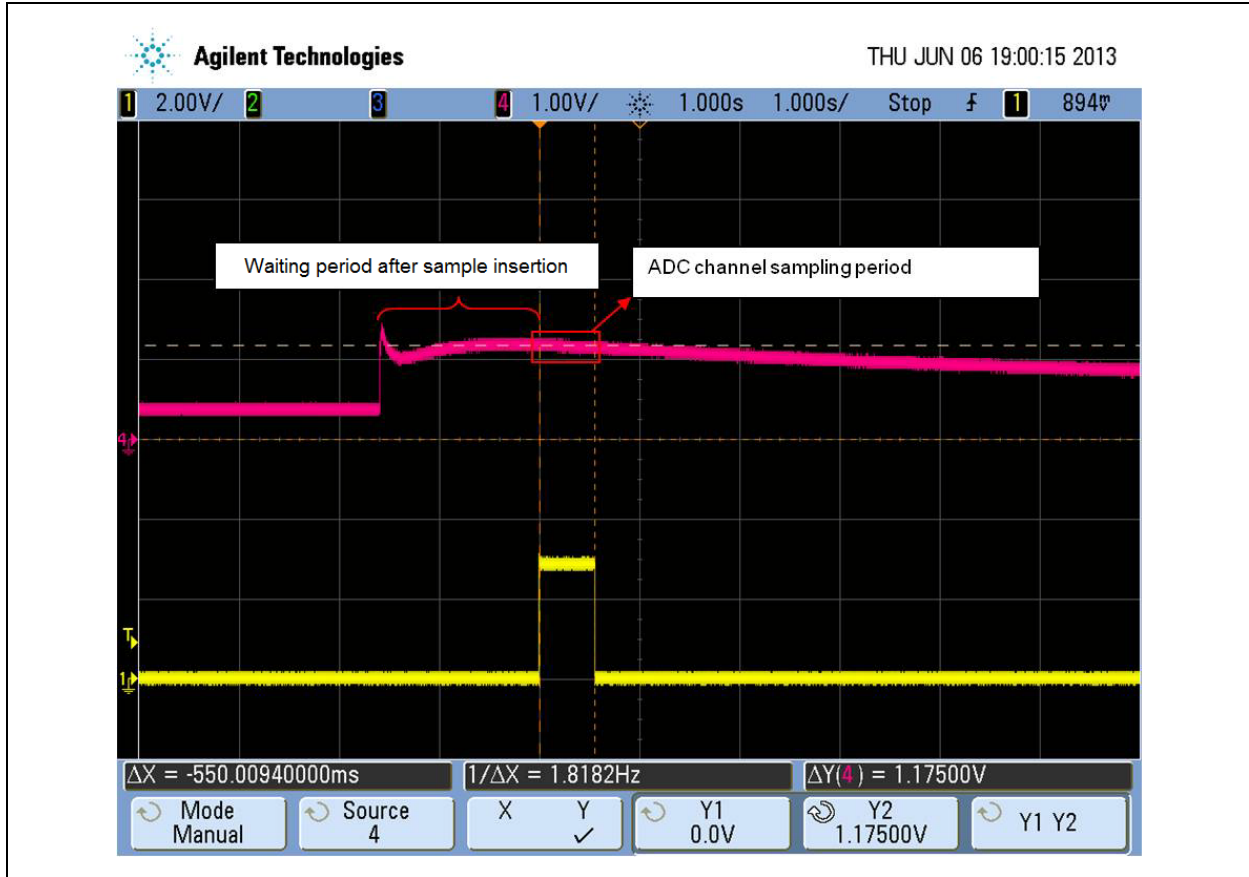


Figure A-2 illustrates the current-to-voltage converter output with normal control solution for the glucose reading of 5.3 mmol/l (i.e., 95.4 mg/dl).


FIGURE A-2: CURRENT-TO-VOLTAGE CONVERTER OUTPUT WITH CONTROL SOLUTION NORMAL

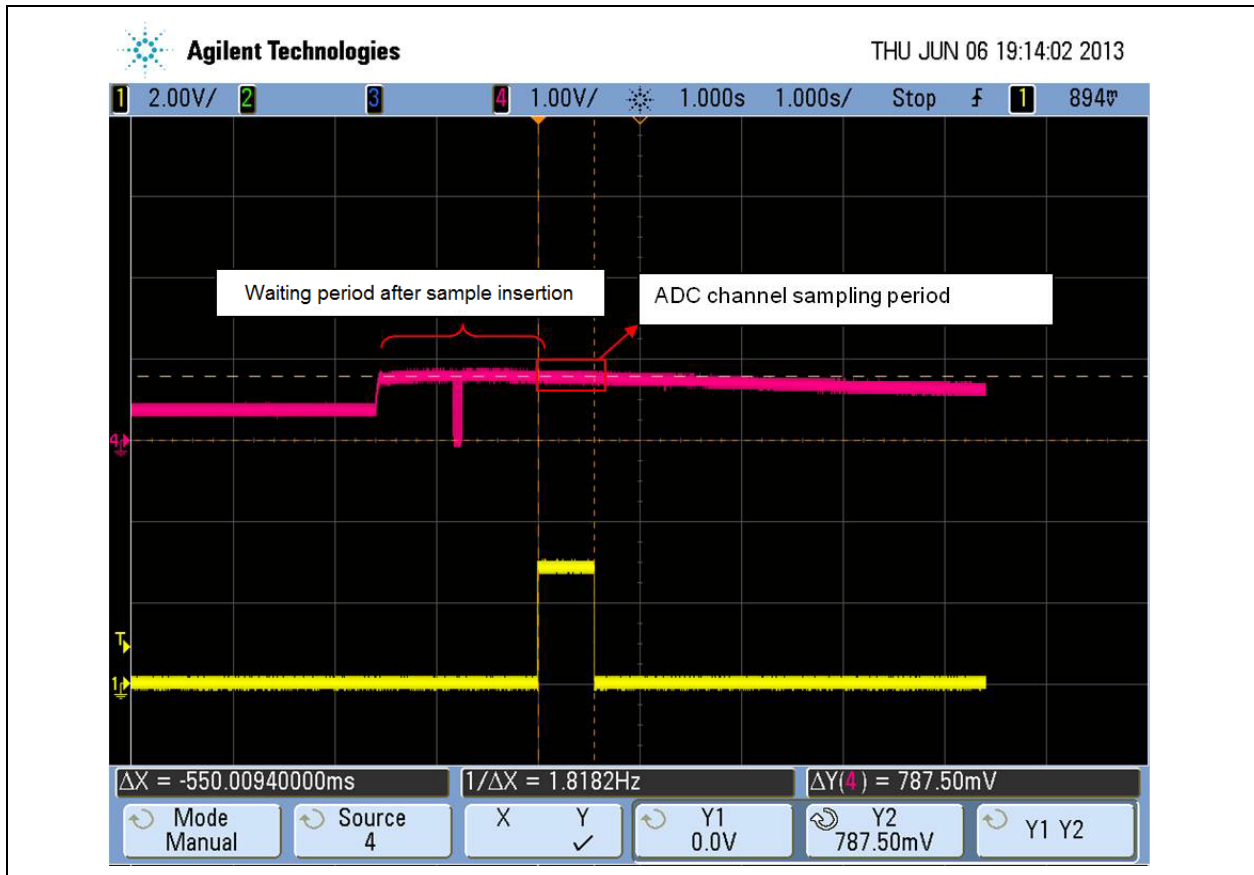

AN1560

Figure A-3 illustrates the current-to-voltage converter output with low control solution for the glucose reading of 1.9 mmol/l (i.e., 34.2 mg/dl).


FIGURE A-3: CURRENT-TO-VOLTAGE CONVERTER OUTPUT WITH CONTROL SOLUTION LOW

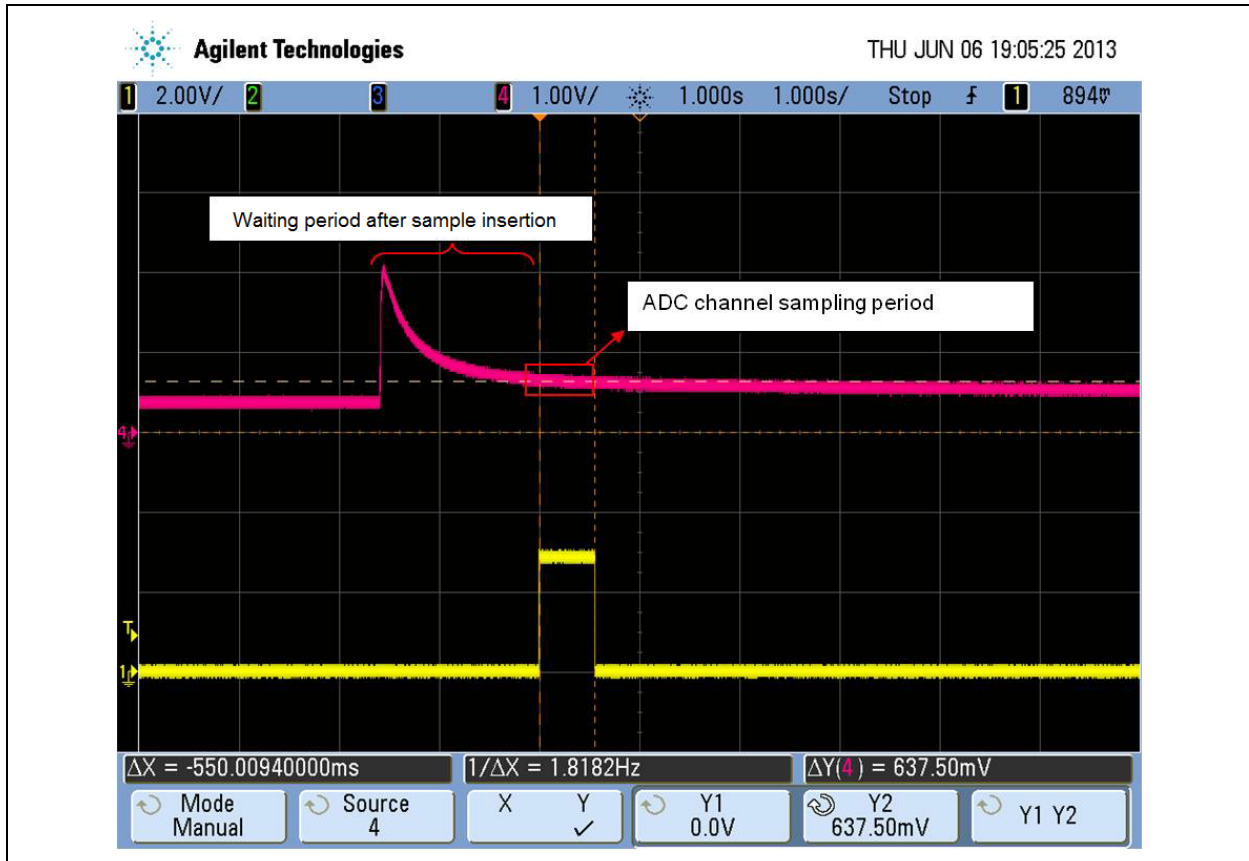

APPENDIX B: GLUCOSE METER REFERENCE DESIGN CIRCUIT SCHEMATICS

Figure B-1 illustrates the glucose meter reference design schematics.

FIGURE B-1: SCHEMATIC

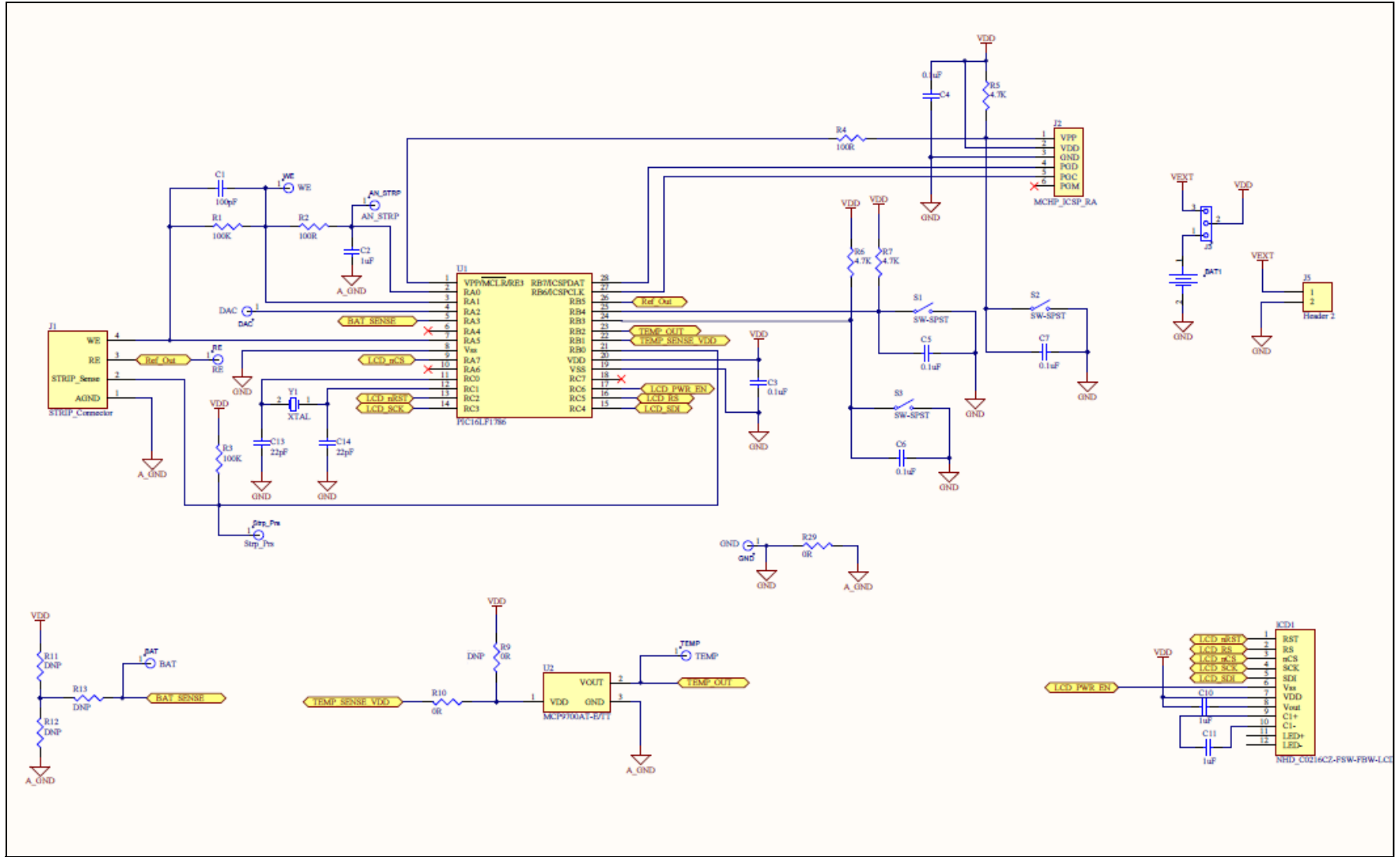


AN1560

APPENDIX C: GLUCOSE METER REFERENCE DESIGN PCB LAYOUT

C.1 PCB Layout Top Layer

Figure C-1 illustrates the top layer of the glucose meter reference design PCB layout.


FIGURE C-1: PCB LAYOUT TOP LAYER

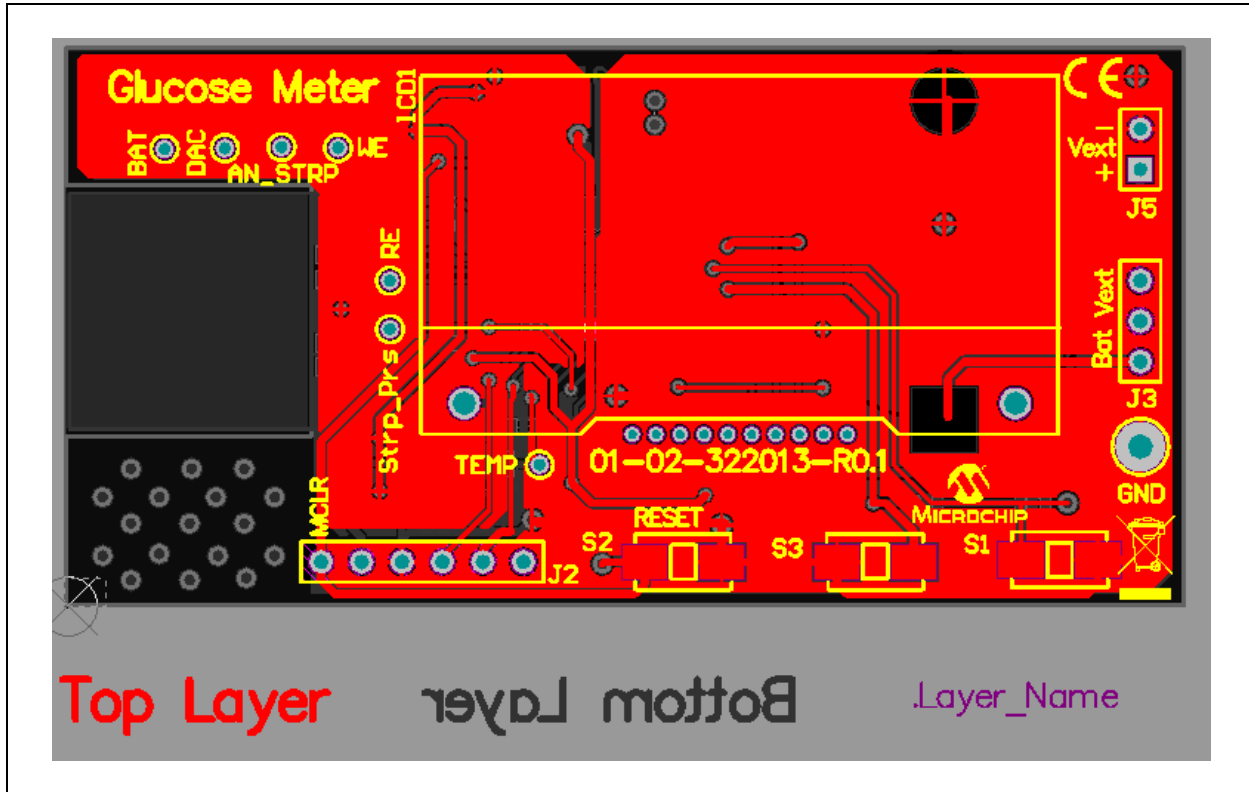

C.2 PCB Layout Bottom Layer

Figure C-2 illustrates the bottom layer of the glucose meter reference design PCB layout.

FIGURE C-2: PCB LAYOUT BOTTOM LAYER

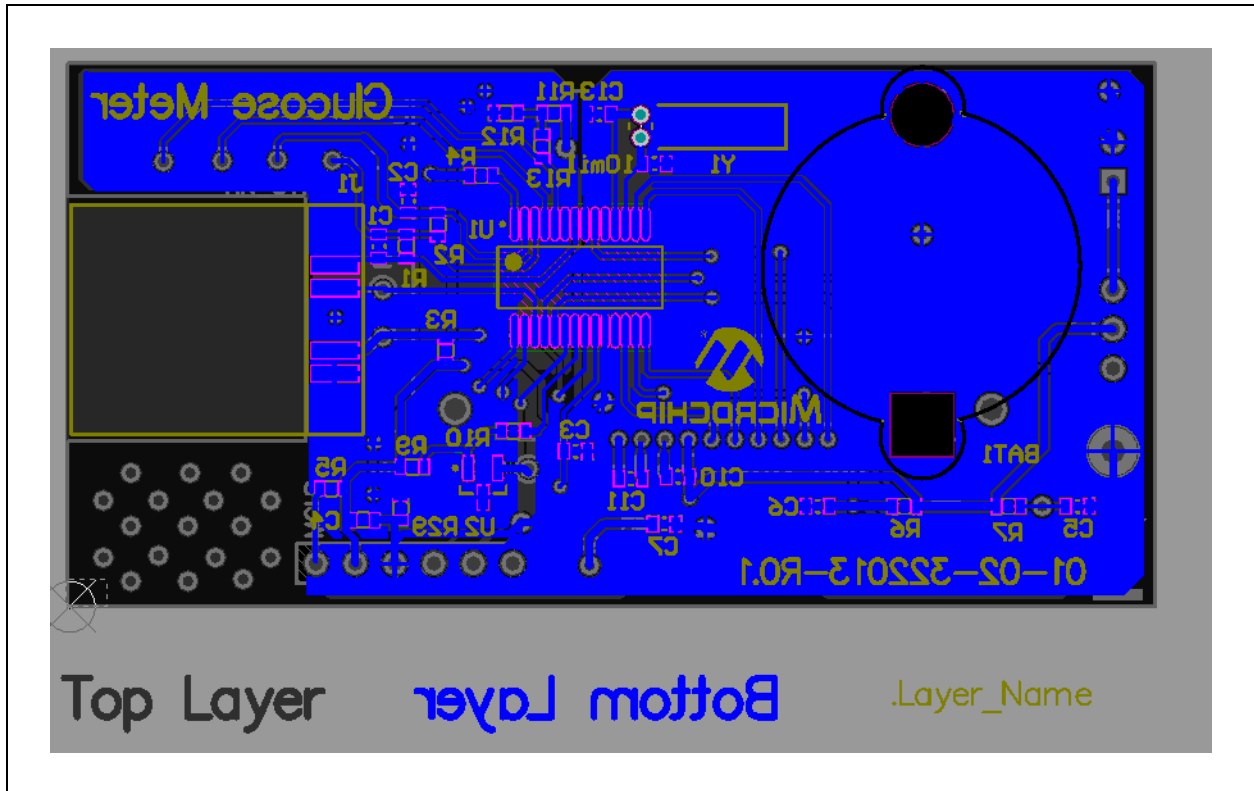

AN1560

NOTES:

Note the following details of the code protection feature on Microchip devices:

- Microchip products meet the specification contained in their particular Microchip Data Sheet.
- Microchip believes that its family of products is one of the most secure families of its kind on the market today, when used in the intended manner and under normal conditions.
- There are dishonest and possibly illegal methods used to breach the code protection feature. All of these methods, to our knowledge, require using the Microchip products in a manner outside the operating specifications contained in Microchip's Data Sheets. Most likely, the person doing so is engaged in theft of intellectual property.
- Microchip is willing to work with the customer who is concerned about the integrity of their code.
- Neither Microchip nor any other semiconductor manufacturer can guarantee the security of their code. Code protection does not mean that we are guaranteeing the product as “unbreakable.”

Code protection is constantly evolving. We at Microchip are committed to continuously improving the code protection features of our products. Attempts to break Microchip's code protection feature may be a violation of the Digital Millennium Copyright Act. If such acts allow unauthorized access to your software or other copyrighted work, you may have a right to sue for relief under that Act.

Information contained in this publication regarding device applications and the like is provided only for your convenience and may be superseded by updates. It is your responsibility to ensure that your application meets with your specifications. MICROCHIP MAKES NO REPRESENTATIONS OR WARRANTIES OF ANY KIND WHETHER EXPRESS OR IMPLIED, WRITTEN OR ORAL, STATUTORY OR OTHERWISE, RELATED TO THE INFORMATION, INCLUDING BUT NOT LIMITED TO ITS CONDITION, QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR PURPOSE. Microchip disclaims all liability arising from this information and its use. Use of Microchip devices in life support and/or safety applications is entirely at the buyer's risk, and the buyer agrees to defend, indemnify and hold harmless Microchip from any and all damages, claims, suits, or expenses resulting from such use. No licenses are conveyed, implicitly or otherwise, under any Microchip intellectual property rights.

Trademarks

The Microchip name and logo, the Microchip logo, dsPIC, FlashFlex, KEELOQ, KEELOQ logo, MPLAB, PIC, PICmicro, PICSTART, PIC³² logo, rPIC, SST, SST Logo, SuperFlash and UNI/O are registered trademarks of Microchip Technology Incorporated in the U.S.A. and other countries.

FilterLab, Hampshire, HI-TECH C, Linear Active Thermistor, MTP, SEEVAL and The Embedded Control Solutions Company are registered trademarks of Microchip Technology Incorporated in the U.S.A.

Silicon Storage Technology is a registered trademark of Microchip Technology Inc. in other countries.

Analog-for-the-Digital Age, Application Maestro, BodyCom, chipKIT, chipKIT logo, CodeGuard, dsPICDEM, dsPICDEM.net, dsPICworks, dsSPEAK, ECAN, ECONOMONITOR, FanSense, HI-TIDE, In-Circuit Serial Programming, ICSP, Mindi, MiWi, MPASM, MPF, MPLAB Certified logo, MPLIB, MPLINK, mTouch, Omniclient Code Generation, PICC, PICC-18, PICDEM, PICDEM.net, PICkit, PICTail, REAL ICE, rFLAB, Select Mode, SQI, Serial Quad I/O, Total Endurance, TSHARC, UniWinDriver, WiperLock, ZENA and Z-Scale are trademarks of Microchip Technology Incorporated in the U.S.A. and other countries.

SQTP is a service mark of Microchip Technology Incorporated in the U.S.A.

GestIC and ULPP are registered trademarks of Microchip Technology Germany II GmbH & Co. KG, a subsidiary of Microchip Technology Inc., in other countries.

All other trademarks mentioned herein are property of their respective companies.

© 2013, Microchip Technology Incorporated, Printed in the U.S.A., All Rights Reserved.

 Printed on recycled paper.

ISBN: 9781620775639

QUALITY MANAGEMENT SYSTEM
CERTIFIED BY DNV
== ISO/TS 16949 ==

Microchip received ISO/TS-16949:2009 certification for its worldwide headquarters, design and wafer fabrication facilities in Chandler and Tempe, Arizona; Gresham, Oregon and design centers in California and India. The Company's quality system processes and procedures are for its PIC® MCUs and dsPIC® DSCs, KEELOQ® code hopping devices, Serial EEPROMs, microperipherals, nonvolatile memory and analog products. In addition, Microchip's quality system for the design and manufacture of development systems is ISO 9001:2000 certified.



MICROCHIP

Worldwide Sales and Service

AMERICAS

Corporate Office
2355 West Chandler Blvd.
Chandler, AZ 85224-6199
Tel: 480-792-7200
Fax: 480-792-7277
Technical Support:
<http://www.microchip.com/support>
Web Address:
www.microchip.com

Atlanta
Duluth, GA
Tel: 678-957-9614
Fax: 678-957-1455

Boston
Westborough, MA
Tel: 774-760-0087
Fax: 774-760-0088

Chicago
Itasca, IL
Tel: 630-285-0071
Fax: 630-285-0075

Cleveland
Independence, OH
Tel: 216-447-0464
Fax: 216-447-0643

Dallas
Addison, TX
Tel: 972-818-7423
Fax: 972-818-2924

Detroit
Farmington Hills, MI
Tel: 248-538-2250
Fax: 248-538-2260

Indianapolis
Noblesville, IN
Tel: 317-773-8323
Fax: 317-773-5453

Los Angeles
Mission Viejo, CA
Tel: 949-462-9523
Fax: 949-462-9608

Santa Clara
Santa Clara, CA
Tel: 408-961-6444
Fax: 408-961-6445

Toronto
Mississauga, Ontario,
Canada
Tel: 905-673-0699
Fax: 905-673-6509

ASIA/PACIFIC

Asia Pacific Office
Suites 3707-14, 37th Floor
Tower 6, The Gateway
Harbour City, Kowloon
Hong Kong
Tel: 852-2401-1200
Fax: 852-2401-3431

Australia - Sydney
Tel: 61-2-9868-6733
Fax: 61-2-9868-6755

China - Beijing
Tel: 86-10-8569-7000
Fax: 86-10-8528-2104

China - Chengdu
Tel: 86-28-8665-5511
Fax: 86-28-8665-7889

China - Chongqing
Tel: 86-23-8980-9588
Fax: 86-23-8980-9500

China - Hangzhou
Tel: 86-571-2819-3187
Fax: 86-571-2819-3189

China - Hong Kong SAR
Tel: 852-2943-5100
Fax: 852-2401-3431

China - Nanjing
Tel: 86-25-8473-2460
Fax: 86-25-8473-2470

China - Qingdao
Tel: 86-532-8502-7355
Fax: 86-532-8502-7205

China - Shanghai
Tel: 86-21-5407-5533
Fax: 86-21-5407-5066

China - Shenyang
Tel: 86-24-2334-2829
Fax: 86-24-2334-2393

China - Shenzhen
Tel: 86-755-8864-2200
Fax: 86-755-8203-1760

China - Wuhan
Tel: 86-27-5980-5300
Fax: 86-27-5980-5118

China - Xian
Tel: 86-29-8833-7252
Fax: 86-29-8833-7256

China - Xiamen
Tel: 86-592-2388138
Fax: 86-592-2388130

China - Zhuhai
Tel: 86-756-3210040
Fax: 86-756-3210049

ASIA/PACIFIC

India - Bangalore
Tel: 91-80-3090-4444
Fax: 91-80-3090-4123

India - New Delhi
Tel: 91-11-4160-8631
Fax: 91-11-4160-8632

India - Pune
Tel: 91-20-3019-1500

Japan - Osaka
Tel: 81-6-6152-7160
Fax: 81-6-6152-9310

Japan - Tokyo
Tel: 81-3-6880-3770
Fax: 81-3-6880-3771

Korea - Daegu
Tel: 82-53-744-4301
Fax: 82-53-744-4302

Korea - Seoul
Tel: 82-2-554-7200
Fax: 82-2-558-5932 or
82-2-558-5934

Malaysia - Kuala Lumpur
Tel: 60-3-6201-9857
Fax: 60-3-6201-9859

Malaysia - Penang
Tel: 60-4-227-8870
Fax: 60-4-227-4068

Philippines - Manila
Tel: 63-2-634-9065
Fax: 63-2-634-9069

Singapore
Tel: 65-6334-8870
Fax: 65-6334-8850

Taiwan - Hsin Chu
Tel: 886-3-5778-366
Fax: 886-3-5770-955

Taiwan - Kaohsiung
Tel: 886-7-213-7828
Fax: 886-7-330-9305

Taiwan - Taipei
Tel: 886-2-2508-8600
Fax: 886-2-2508-0102

Thailand - Bangkok
Tel: 66-2-694-1351
Fax: 66-2-694-1350

EUROPE

Austria - Wels
Tel: 43-7242-2244-39
Fax: 43-7242-2244-393

Denmark - Copenhagen
Tel: 45-4450-2828
Fax: 45-4485-2829

France - Paris
Tel: 33-1-69-53-63-20
Fax: 33-1-69-30-90-79

Germany - Munich
Tel: 49-89-627-144-0
Fax: 49-89-627-144-44

Italy - Milan
Tel: 39-0331-742611
Fax: 39-0331-466781

Netherlands - Drunen
Tel: 31-416-690399
Fax: 31-416-690340

Spain - Madrid
Tel: 34-91-708-08-90
Fax: 34-91-708-08-91

UK - Wokingham
Tel: 44-118-921-5869
Fax: 44-118-921-5820

08/20/13