


1. Tape the cannula down to this paper lightly over the illustration.
2. Cut a strip of duct tape 1/4" by 4 1/4".

3. Place strip of tape across cannula just above the nostril tubes adhesive down (light gray below) Fold the ends around the cannula to cover all adhesive.
4. Detach cannula from the paper and flip it over horizontally.
5. Cut a second strip of duct tape 1/4" by 4 1/4" and place it over the first piece Covering all of the exposed adhesive of the first strip of duct tape and wrap the ends again, hiding all remaining adhesive.
6. Wear cannula as usual, but with the tape draped across the bridge of the nose. The cannula will resist sliding sideways now.

